



Metal spring cartridge Silicone spring cartridge Biolock cartridge

The Biolock Cartridge Advantage

SkinStylus® Gold system use the patent pending Biolock cartridge which has been engineered with a seven element safety system that greatly reduces the risk of cross contamination. The Biolock cartridge incorporates a labyrinth design with absorbent membranes and liquid expelling ports to ensure fluids cannot leave the cartridge. The Biolock cartridge is screw-affixed to ensure that it cannot come off unexpectedly during a procedure and to also snugly seat the cartridge into a removable, aluminum, nose cone. The seated end of the Biolock cartridge has a female connector with a silicon gasket that engages and seals off the drive piston of the micro device. In this way, any fluids that could have exited the cartridge would be forced away from the sealed drive piston of the micro device and become trapped inside the nose cone. The SkinStylus® was designed to have the nose cone removed and autoclave (or otherwise) reprocessed after each procedure. The Biolock cartridge is one of the only cartridges that is gamma ray sterilized which offers true sterility upon initial use.

Although, the outside casing may look similar to other micro devices, SkinStylus® MicroSystems incorporate the latest advances in motor efficiency, battery life, and the flexibility of both corded and cordless power systems. As discussed, SkinStylus® incorporates the Biolock™ cartridge to prevent liquids from entering into the microdevice. More importantly, SkinStylus® is the ONLY microsystem that can be autoclave reprocessed! The components were carefully selected and our one year warranty speaks to our confidence in the quality of our design. As busy practitioners, we always want to be prepared and not have to turn a client away. SkinStylus® MicroSystems are carefully designed to incorporate all of the essential features that busy skin care professionals should expect from a professional system, all at an affordable price point.



**Biolock™ 13 Needle
Cartridge**



Anatomy of a Professional MicroSystem

The Gold SkinStylus® QuickStart Kit

The SkinStylus® MicroSystem can be reprocessed in an autoclave sterilizer! Other systems can trap biohazardous fluids inside the device that cannot be removed or reprocessed in an autoclave. Don't put your clients and yourself at risk of cross contamination! The SkinStylus® Biolock cartridge has a unique, patent pending, seven step safety system designed to stop any fluids or ink from getting through the cartridge and into the device. The Biolock cartridge is also gamma ray sterilized to ensure client safety.

Redundant Safety Systems: SkinStylus® practitioners can rest assured that even in the unlikely event that fluids or ink were somehow able to exit the Biolock cartridge, the SkinStylus® nose cone can be reprocessed in an autoclave after each use further minimizing the risk of cross-contamination.



The Gold SkinStylus® QuickStart Kit is packaged in an attractive protective case and includes:

- 1 Gold SkinStylus® motor
- 2 Li-on Batteries
- 2 autoclave sterilizable nosecones
- Corded module with black cord/block(allows either stylus to be operated in corded mode)
- White Battery Charging cord/block
- 20 Bioclock Cartridges
- 20 SkinStylus® brochures
- HA-Cu Serum 30 ml
- 1 year warranty